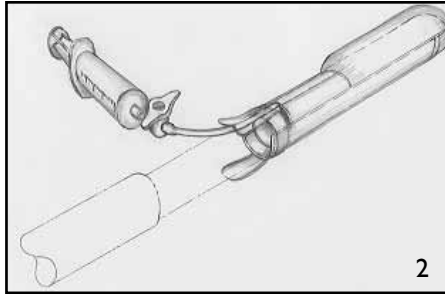
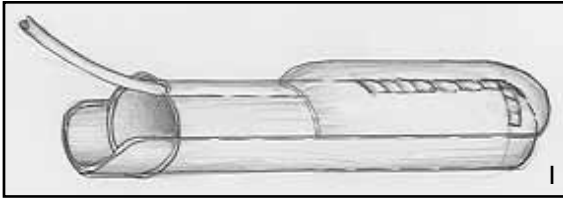
Endocavity Balloon

REFERENCE GUIDE

CE
0120

INTENDED USE

- The transducer standoff device can be used to perform scanning and needle guided procedures for body surface and endocavity diagnostic ultrasound.

WARNING

- In the USA, federal law restricts device to sale or use by or on order of a physician.
- Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.

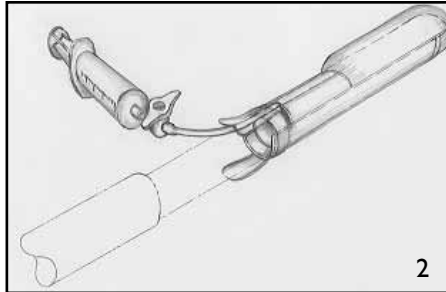
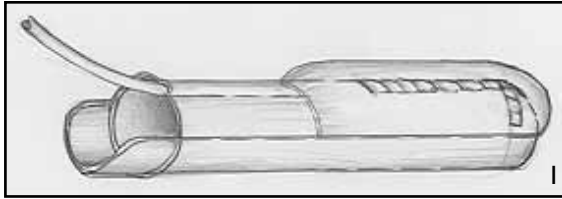
Applying Balloon to Transducer

1. Place an appropriate amount of gel inside endocavity balloon and/or on transducer face to aid in scan quality, application and removal of balloon.
2. Insert transducer into balloon with fill tube oriented in top-center of transducer. Pull balloon tightly over transducer face to remove wrinkles and air bubbles.
3. Visually orient balloon so lateral seams are symmetrical distance from longitudinal array.
4. Tape balloon securely making sure to attach around plastic tabs (Tape not included).
5. Fill syringe with 30cc of saline solution. Fill balloon through luer lock port holding probe tip down; fill and aspirate with 10-20cc until all air is removed. Do not over inflate balloon.
6. Deflate balloon leaving 25-30cc syringe connected.

Disposal

WARNING

- Dispose of single-use components as infectious waste.

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COPYRIGHT © 2010 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. U.S. PATENT NO. 6,126,607. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH