Grid Adapter


REFERENCE GUIDE

CE

Grid Adapter

.....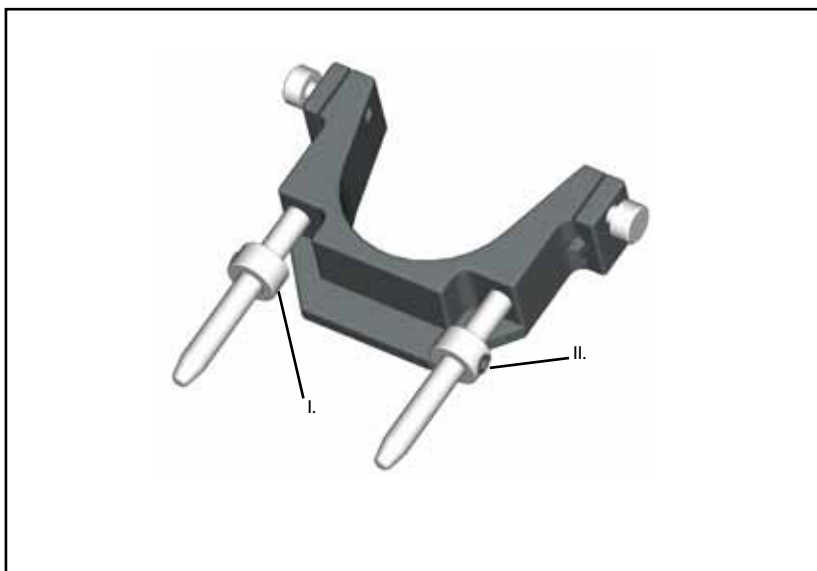

I. Adjustable Collars

Indstillelige kraver

Instelbare kragen

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Colliers ajustables

Einstellbare Schellen

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Ayar Vidası

INTENDED USE

- The grid adapter is designed to position a grid with respect to an endocavity ultrasound probe.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Ensure graphics on ultrasound monitor match graphics on grid.*

Using Grid Adapter

1. Calibrate height using disposable grid for your ultrasound system.
2. Securely tighten adjustable collar set screws at required elevation with wrench provided.

Cleaning, Disinfecting and Sterilizing

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
 - *Do not autoclave.*
1. Remove grid adapter from stepping device.
 2. Clean surfaces with mild detergent, such as ENZOL[®] Enzymatic Detergent (Johnson & Johnson) following the manufacturers recommendation for use.
 3. Disinfect all surfaces with common germicidal or antiseptic solution, such as alcohol or hydrogen peroxide.
 4. Allow components to air dry.

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COPYRIGHT © 2010 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH