Viewing Wand Cover

REFERENCE GUIDE

CE
0120

Viewing Wand Cover

English

INTENDED USE

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intraoperative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

WARNING

- In the USA, federal law restricts device to sale or use by or on order of a physician.
- Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.
- This product contains natural rubber latex, which may cause allergic reactions. In the USA, refer to FDA Medical Alert MDA91-I.
- Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.
- Refer to Viewing Wand Operator's Guide for instructions for use.
- Dispose of single-use components as infectious waste.

1. Insert transducer into cover making sure to use proper sterile technique. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover.
2. Secure cover with enclosed bands.
3. Inspect cover to ensure there are no holes or tears.

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COPYRIGHT © 2009. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH